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Instructions for use

The Product
Revamil® Balm
15/50 g ointment for the treatment of superficial wounds
Ingredients
Peanut oil refined, beeswax white, Glyceryl oleate, honey, purified water

Product description
Revamil® Balm is a nourishing ointment for the treatment of superficial wounds. Revamil® Balm consists of a base ointment in combination with pure honey. The honey in Revamil® Balm contains high levels of honey enzymes, like glucose oxidase. In a moist environment, this enzyme produces low levels of hydrogen peroxide. The high sugar concentration of honey facilitates mixing of wound moisture with Revamil® Balm, thus forming an intermediate layer between the wound and the honey, which gives anti-bacterial protection. Revamil® Balm does not contain residues from pesticides, herbicides, conservatives or other components that are known to cause allergic reactions.

Instructions for use

- Apply a thin layer of Revamil® Balm on the wound
- The wound can be covered with a wound dressing
- Treat affected areas once every 24 to 48 hours;
- In case of persisting itching, Revamil® Balm can be applied more frequently

Indications
Revamil® Balm can be applied on superficial wounds. Revamil® Balm is allowed for all age groups

Contra indications
Do not use the product when the patient is known to be sensitive to one of the ingredients of the ointment (see Ingredients). Hypersensitivity to one of the ingredients of the ointment can cause redness, irritation and possibly light pain.

- Side effects**
 - During the first minutes after application, the product can cause a light stinging sensation. The stinging sensation is probably caused by the low pH and high osmolality of the honey and has no effect on the performance of the product.
 - In the event of unexpected side effects after use, contact your doctor or the manufacturer. Side effects can cause inconvenience to the patient.

- Safety precautions**
 - Do not use if the tube is damaged as this can lead to contamination of the product with bacteria and to wound infection. Because of the antibacterial protection that honey provides, the chance of wound infection is considered very low.
 - Always replace the top on the tube after use. An open tube could lead to contamination with bacteria and wound infection. Because of the antibacterial protection that honey provides, the chance of wound infection is considered very low.
 - Do not allow the tube to come into direct contact with the surface of the wound. Direct contact between the wound/ skin and the uncappped tube could lead to contamination with bacteria and wound infection. Because of the antibacterial protection that honey provides, the chance of wound infection is considered very low.
 - Do not apply this product for more than 30 days continuously. Treatment for more than 30 days continuously could lead to inconvenience to the patient, such as irritation and/or minor pain.
 - The contents of the tube must be used within 3 months of opening. Longer use could lead to lower functionality and contamination of the product. Because of the antibacterial protection that honey provides, the chance of wound infection is considered very low.
 - Do not treat different patients with the same tube as this can lead to cross-contamination with bacteria and to wound infection. Because of the antibacterial protection that honey provides, the chance of wound infection is considered very low.
 - Do not use the product after the expiration date as this can lead to lower performance of the product and to inconvenience for the patient.

Storing conditions
Store between 4°C and 30°C

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Product
Revamil® Balm
15/50 g Salbe zur Behandlung von oberflächlichen Wunden
Ingrediënten
Arachis oleum, cera alba, glyceryl oleate, mel, aqua

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Bfactory Health Products B.V., Remmerden 58, 3911 TZ Rhenen (NL)
DK: Axel Madsen A/S, Vedbæk